# Space Chamber

## FOR MULTIPLE PATIENT USE

### Indications for use

The device is intended to be used by patients to administer aerosolised medication from most pressurised metered dose inhalers. The intended environments for use include home, hospitals and clinics.

### Priming your Space Chamber

Before using the Space Chamber for the first time it will need to be primed. To prime the Space Chamber, immerse in warm water with a mild liquid detergent. **Do not** rinse as detergent minimises static charge and allow to air dry. Alternatively, the Space Chamber may be primed by depressing your inhaler 3 - 4 times into the Space Chamber.

Your Space Chamber is now ready to use.

Your Space Chamber will need to 'primed' again if it is only used occasionally.

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## How to use the Space Chamber

1

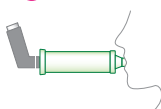
Remove the cap from your inhaler and shake well.

2

Push the inhaler into the end cap of the Space Chamber.

3

Put the mouthpiece between the lips and make an airtight seal, depress your inhaler, you will see a fine spray inside the Space Chamber.

Take a slow, deep breath through the mouth and hold for 5-10 seconds or continue to breathe in and out through the Space Chamber for 4-6 breaths.

4

Wait at least one minute before repeating step 3. Only use one dose from the inhaler at a time.

5

If a face mask is needed, attach it to the mouthpiece of the Space Chamber. Ensure the connection is secure at all times.

Apply to face and check for effective seal.

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## Please note

- Always follow the instructions provided with your inhaler.
- Before use check for foreign objects, broken or deteriorated parts.
- Replace Space Chamber immediately if any damage or missing parts are evident.
- Check to see that the valve in the mouthpiece moves when you breathe in and when you breathe out.
- All components used in the Space Chamber are not made with natural rubber latex.
- The device may form production related stress cracks if coming into contact with rinse aids. If using automated washing as a part of the cleaning and autoclaving process, the device must be tested for compatibility.

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## FOR MULTIPLE PATIENT USE

### Cleaning

The Space Chamber should be cleaned at least once a month but can be washed more often if needed.

1. Remove the end cap by placing your thumb in the hole then pull and twist at the same time. There is no need to dismantle the mouthpiece.
2. Wash all parts in warm water with a mild liquid detergent.
3. **Do not rinse** as detergent minimises static charge.
4. Shake out excess water and allow to air dry. **Do not rub dry.**
5. Firmly replace end cap once the Space Chamber is dry.
6. Prime the Space Chamber as described overleaf.

The Space Chamber can be cleaned in a dishwasher but must be primed again as described overleaf.

NOTE: Avoid rinse aids.

If it is not clear to you on how to clean your Space Chamber please ask your healthcare professional for help.

Replace within 12 months.

### Autoclaving - Multiple Patient Use

The Space Chamber should be autoclaved between patients.

1. Clean the Space Chamber as per the above cleaning instructions.  
NOTE: Avoid rinse aids if using automated washing to clean.
2. Disassemble the Space Chamber fully as illustrated below.
3. If washing the Space Chamber prior to autoclaving, it must be rinsed free of all detergent.
4. The Space Chamber has been tested to meet the minimum Australian Standards of 134°C for 3 minutes.
5. Wrap the Space Chamber in a cloth or two layers of 1-ply polypropylene wrap (Eg:KC600) to ensure that it does not come into direct contact with the metal parts of the autoclave.
6. **Do not** place other objects on the Space Chamber.  
– Autoclave at 134°C for 3 minutes.  
– Recommended cooling time is 30 minutes.
7. Reassemble the Space Chamber as illustrated below. Ensure the mouthpiece clicks into place.
8. Submerge and agitate in warm water mixed with mild liquid detergent.
9. **Do not rinse**, as the detergent helps to reduce the static charge.
10. Shake out excess water. **Do not rub dry.** Allow to drip/air dry.

**Note:** Space Chamber can be autoclaved up to 100 times.

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**Medical Developments International Limited**
Distributed by: CAREstream Medical Ltd.